



The Answer to a Quicker Recovery



 Manufacturer
Trice Medical
40 General Warren Blvd, Suite 100
Malvern, PA 19355, U.S.A.
610.989.8080
www.tricemedical.com

4-10-0085 Rev B

 EC REP
mdi Europa GmbH
Langenhagener Str. 71
D-30855 Langenhagen
Germany



mi-eye 2[®]

CARE INSTRUCTIONS

By choosing mi-eye 2[®], you have taken the first step to getting back to your active life sooner than when choosing an MRI*. With the help of mi-eye 2[®] you and your physician can now make a decision on what treatment is best for YOUR unique condition.

What's Next?

Following your mi-eye procedure, you will have to take care of the procedure site, as it is an open wound. You may also experience some aches, pains and drainage from the site. This is normal.

To make sure that it heals properly, follow the Care Instructions and any special instructions from your physician.

Follow Trice Medical on social media
and share your mi-eye 2[®] story.



*Based on estimated timeframe for radiology testing and results, including follow-up consultation.

Care Instructions

If you experience post-procedure aches or pains once the local anesthesia has worn off, you should first follow the **RICE** technique—**Rest, Ice, Compression and Elevation**. If pain remains unresolved, you can take over-the-counter anti-inflammatory medications, such as an NSAID (ibuprofen, aspirin, etc.) or acetaminophen in accordance with labeled instructions. Be sure to follow your doctor's recommendations for the right course of at-home pain management, including which type of medication is best for you. If any chills, fever or warm sensation at the procedure site are experienced, or any other adverse effects, contact your physician as soon as possible.

Bandage

Prior to leaving the physician's office, a bandage was placed over the mi-eye procedure site. Any post-procedure bleeding should resolve quickly. If prescribed by your doctor, the procedure site can be left open to air within a day of the procedure. If continued drainage from the procedure site is observed, you can re-apply a bandage to the site. Contact your physician if drainage appears abnormal or seems excessive.

Returning to Activity

Unless you experience any lingering symptoms, you can return to activity following a mi-eye procedure. Be sure to follow any recommendations for restricted activity your physician may have advised.

Don'ts

It is recommended to refrain from submerging the area until it has fully healed to prevent potential infection. You may shower the area as soon as you would like, unless otherwise directed by your physician.

mi-eye 2[®] is for use in diagnostic and operative arthroscopic and endoscopic procedures to provide illumination and visualization of an interior cavity of the body through either a natural or surgical opening, and is available by prescription only. Talk to your doctor to learn more about mi-eye 2[®]